


Support Rail with Concealed fixings

Fitting and user instructions

Congratulations on purchasing your support rail.

The Support Rail offers a safe hand hold providing:

- * a support to push or pull against when standing up or sitting down.
- * a steadying support & balance when standing or dressing.
- * a firm grip whilst transferring from one position to another.
- * aesthetically pleasing concealed fixings.

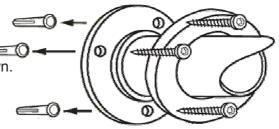



Fitting

WARNING: Before fitting, ensure that there are no concealed electrical cables or pipes where the fixing holes are to be drilled. If installing a metallic rail in a bath/shower area the rail may need to be earthed (IEE wiring regulations); not all rails are suitable for use in a wet area.

- Ensure the wall and fixings being used are suitable. Nymas are unable to specify fixings due to the number of fixing/wall type combinations.
- 2. Position the Support Rail on the wall either horizontally or at an angle to suit and mark the hole locations.
- 3. Drill the holes to the correct depth for the wall and fixing type used.
- 4. Ensure the rail is securely fastened to the wall and fit flange covers to conceal the fixings.

If in doubt about anything on this sheet consult a suitably qualified professional.

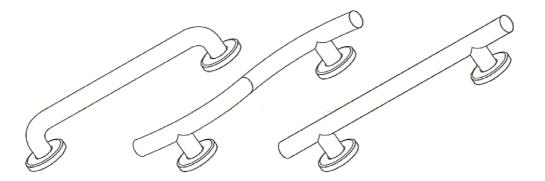

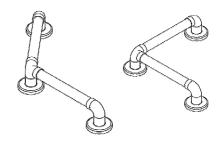

Using The Support Rail

- Before first use, check that the Support Rail is securely fitted to the wall.
- * Use the Support Rail to assist; **NEVER** apply your full weight to the support rail.
- * Regularly clean the Support Rail with a non-abrasive propriety cleaner.
- * All fixings and bolts should be checked on a regular basis by a competent person.

Please note: Any fixings supplied in this package will not suit every type of wall. Consult a qualified tradesman before fitting.

Bristan, Birch Coppice Business Park, Dordon, Tamworth, B78 1SG Telephone: 0844 701 6274 Facsimile: 0844 701 6275

Web Site: www.bristan.com Email: enquire@bristan.com







